

**0.9% Sodium Chloride Injection, USP**  
**Aquastat™ Saline Flush Syringe**

**Caution: Federal (USA) law restricts this device to sale by or on the order of a physician**

**Description:**

**Aquastat™** Saline Flush Syringe contains sterile, non-pyrogenic, preservative free 0.9% Sodium Chloride Injection, USP in a plastic, disposable single use syringe. No latex used in the manufacture of this device.









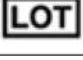



**Fluid and Fluid Path are Sterile. Do NOT place on Sterile Field.**

**Indications and Usage:**

Aquastat™ Saline Flush Syringes are intended for the flushing of compatible intravenous administration sets and indwelling vascular access devices.

**Contraindications:** None

**Symbol Legend:**

	Manufacturer		By Prescription Only		Temperature limitation
	Catalog Number		Single Use Only		Caution
	Consult Instructions for Use		Do not use if pack is opened or damaged		Lot Number
	Use by Date		Do not re-sterilize		Sterilized by Irradiation



**Caution!** Caution statements describe conditions or actions that can result in damage to the product

**Cautions:**

1. Do not use if package is not intact.
2. Do not use if tip cap is improperly affixed or dislodged from syringe tip or syringe is damaged, exhibiting signs of leakage.
3. Do not use syringe if upon visual inspection solution is discolored, cloudy or hazy, contains a precipitate, or has any type of particulate matter.
4. Verify expiration date. Do not use if product has expired.

**Adverse Events:** None.

**Storage:** Store at 20-25°C (68-77°F) excursions permitted to 15-30°C (59-86°F); Do not Freeze.

**Directions for Use:**

1. Utilize proper aseptic technique.
2. Open package by tearing pouch gently.
3. Remove syringe cap by twisting off.
4. If required, attach blunt cannula or needle to syringe following applicable manufacturer and institution guidelines.
5. Hold syringe upright and expel air if present.
6. Attach to needleless system or IV access device and flush adhering to institution's policies and indwelling device manufacturer's recommendations.
7. Discard unit after single use. Do not reuse.

**How Supplied:**

Aquastat™ Saline Flush Syringes are provided in cartons containing individually packaged syringes. Scan QR code with an appropriate scanner for a digital copy of the Direction Insert.

Reorder No. 2T0807 – 03mL in 12mL syringe – 400/case

Reorder No. 2T0805 – 05mL in 12mL syringe – 400/case

Reorder No. 2T0806 – 10mL in 12mL syringe – 400/case

Reorder No. 2T0806-SP – 10mL in 12mL syringe – 120/case